


Patient PPE Mask Instructions

April 5, 2020 | Emiley Joyce

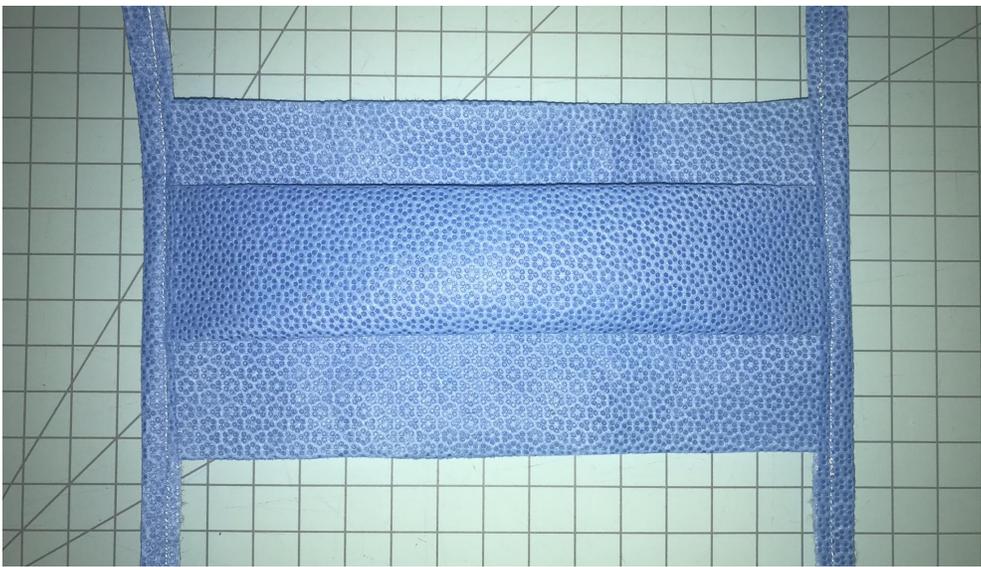

Materials & Tools

Materials:

- Fabric - Sterilization Wrap
 - 1 Ply Design H300-H600
 - 2 Ply Design H100-H200
- Straps - Sterilization Wrap
- Thread




Tools:

- Sewing Machine
- Scissors
- Clips



Pattern (1 Ply)

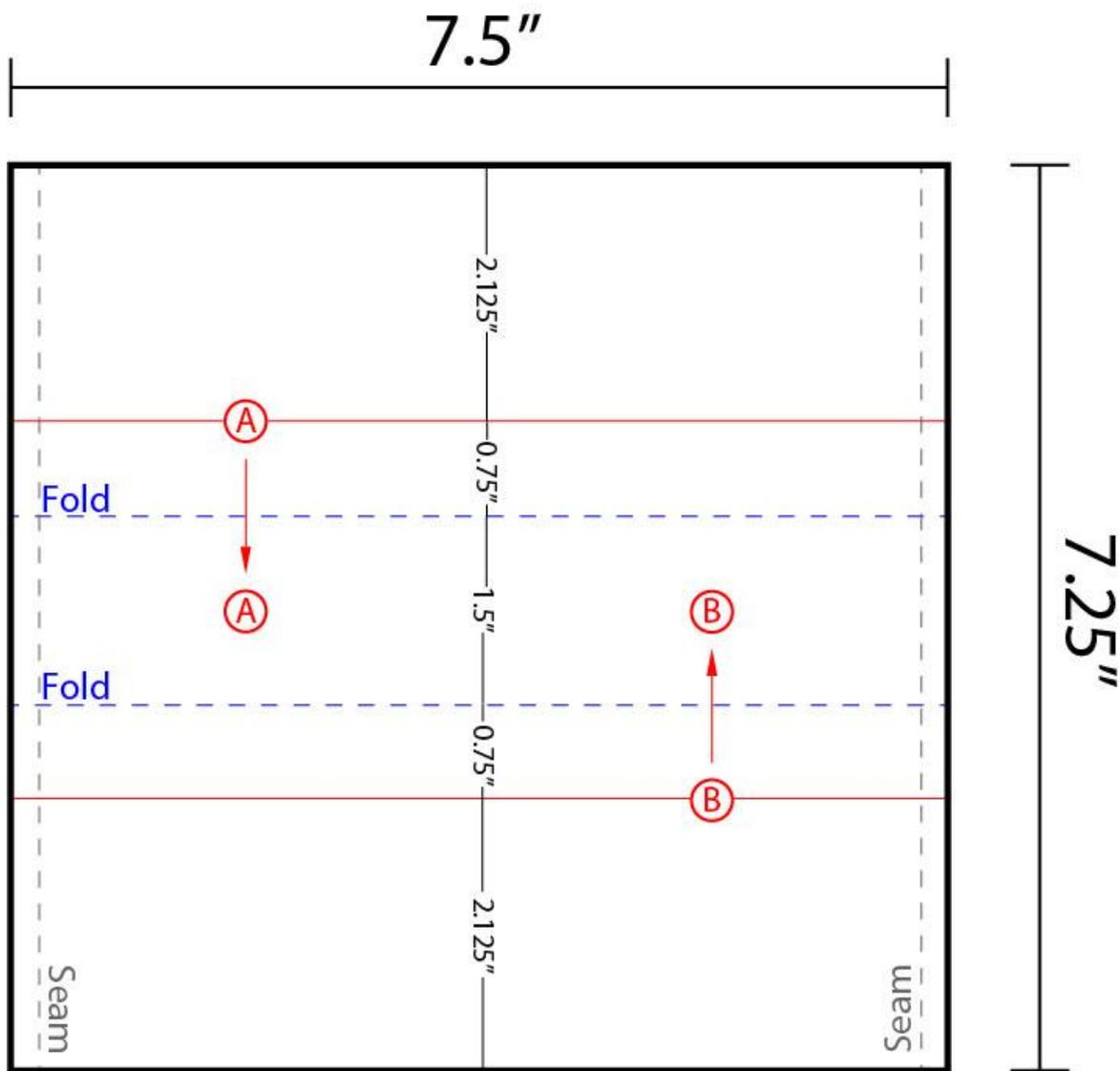

4 Straps

OR




2 Straps

$\frac{3}{4}$ " X $17 \frac{1}{2}$ "

$\frac{3}{4}$ " X 34"


Instructions (1 Ply)

1. Cut a 7.5" x 7.25" piece of fabric.
2. Mark (edges only) and hand press pleats according to the provided pattern.
3. Clip pleats to secure before sewing seams.
4. Sew a 1/4" seam on both ends to hold the pleats in place.

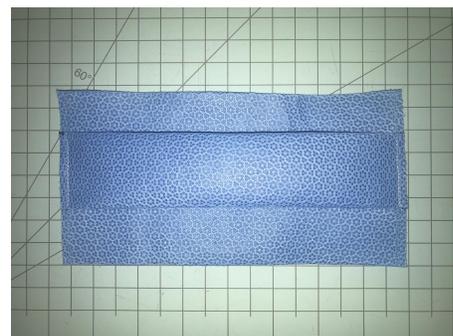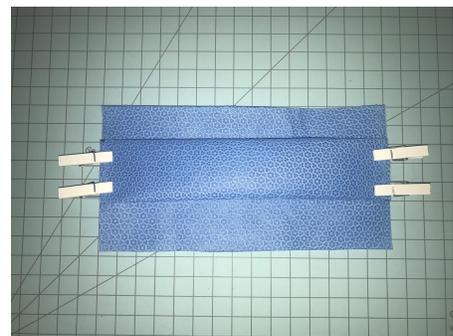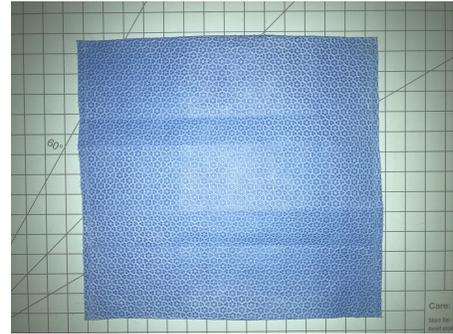


Instructions (1 Ply)

6. Fold one strap piece front to back around raw edge and sew. (2 straps)

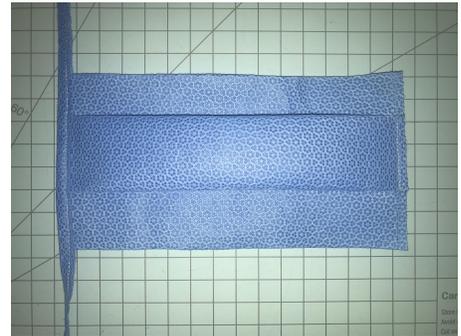

7. Repeat step 6 for the remaining strap.

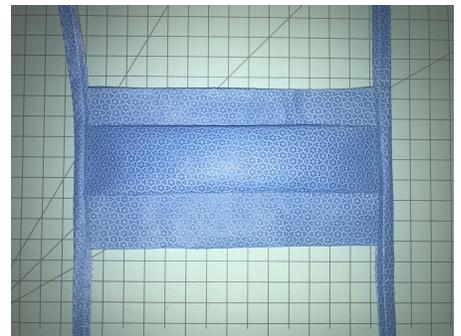

Pattern (2 Ply)


4 Straps

$\frac{3}{4}$ " x $17 \frac{1}{2}$ "

OR


2 Straps

$\frac{3}{4}$ " x 34"

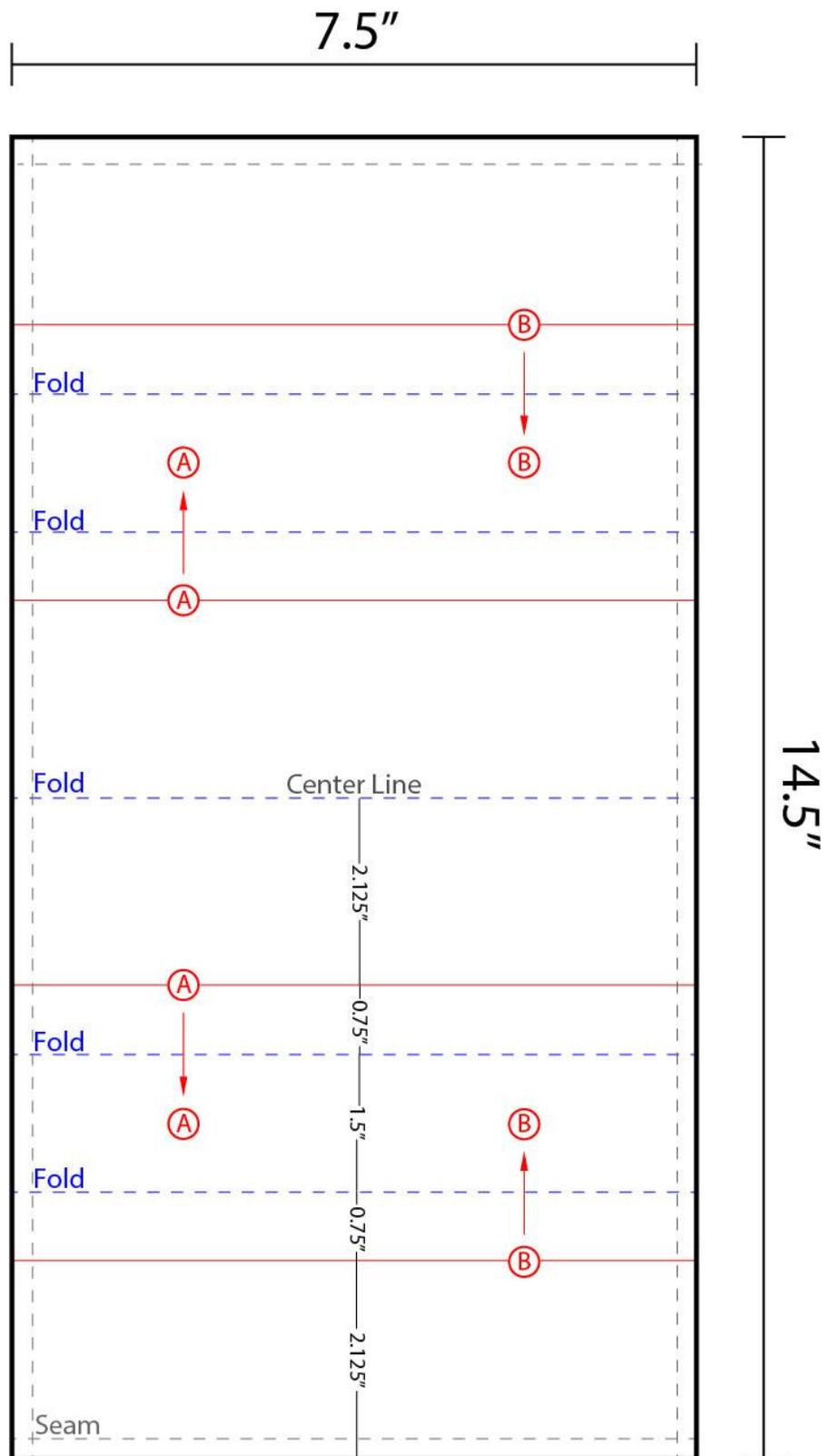

Instructions (2 Ply)

1. Cut a 7.5" x 14.5" piece of fabric.

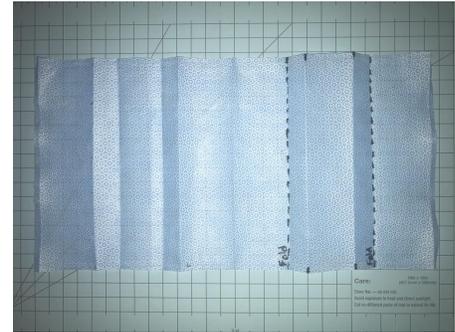

2. Fold fabric in half and sew a 1/4" seam on non-folded end.



3. Turn inside out so the seam is inside and hand press to flatten.



4. Mark (edges only) and hand press pleats according to the provided pattern.



Instructions (2 Ply)

6. Clip pleats to secure before sewing seams.



7. Sew a 1/4" seam on both ends to hold the pleats in place.



8. Fold one strap piece front to back around raw edge halfway up the mask and sew. (4 straps)



9. Repeat step 8 for the remaining straps. Overlap straps on the same side of the mask.



Disclaimer

This document is provided for general use only. The purpose of this document is to demonstrate a method for making emergency masks for UNC Health's use. The use of these instructions outside of UNC Health is at the sole risk of the maker and user. We do not provide any assurances of efficacy or testing in accordance with FDA regulations. Therefore, the Joint Department of Biomedical Engineering at UNC and NC State, UNC Health, and FastTraCS expressly disclaims any warranties of merchantability, any defects in design or production, and any liability for any injuries or other claims for its supportive manufacturing or plan/pattern distributing role in any Personal Protection Equipment (PPE) devices, accessories, components, and/or parts.

